

# BUSTER Body Suit Step'n Go for Cats

Try an ideal alternative to E-collars for cats recovering from surgery or minor injuries. The soft and breathable body suit allows the cat to move freely.

It provides optimal protection and comfort while preventing the patient from reaching tender areas, hence promoting faster healing and recovery.

### Perfect for

- Postoperative care
- Wound care
- Neuter recovery
- Hot spots and skin diseases

### Enables

- Easy fitting
- Freedom of movement
- Stressless handling
- Insertion of pads for fluid-control

### Provides

- Fast recovery
- Less stress
- Mental well-being
- Comfortable fit

### Instructions for Use

1. Place the body suit inside out on a flat surface
2. Insert the front and hind legs separately into the openings
3. Secure the touch fasteners and press studs along the back of the cat
4. Place the closing flap between the rear legs and insert the tail through the tail opening
5. Attach the closing flap to the touch fasteners on the lower back and secure the press studs



Scan QR code to watch the video

Cat. No	Description	Pk Size	Body Length	Chest Circumference	Abdomen Circumference
273946	BUSTER Body Suit Step'n Go for Cats, XXXS	1/pk	27.5 cm / 11"	36 cm / 14"	30 cm / 12"
273947	BUSTER Body Suit Step'n Go for Cats, XXS	1/pk	33 cm / 13"	42 cm / 17"	36 cm / 14"
273948	BUSTER Body Suit Step'n Go for Cats, XS	1/pk	38.5 cm / 15"	46 cm / 18"	40 cm / 16"
272949	BUSTER Body Suit Step'n Go for Cats, S	1/pk	43.5 cm / 17"	52 cm / 20"	46 cm / 18"
273950	BUSTER Body Suit Step'n Go for Cats, Starter Pack, XXXS-S, 4/pk				